

Walk-in Bath

User instructions

Welcome

Congratulations on your purchase of a walk-in bath tub.

Your bath is a straightforward unit to use, but we would ask that you take the time to read these instructions carefully prior to use to ensure that you get the best out of it.

Instructions for use

1. Operate the door lock handle to release the door. The door may now be opened.
2. Enter the bath through the open doorway. Take care not to lean or place any weight on the door as you enter the bath.
3. Close the door. Operate the door lock handle so that the bath door is firmly locked shut.
4. Operate the faucets and fill the bath to the desired level.

Safety notice: Users should take care not to scald themselves (and in particular their legs and feet) with hot water from the hot tap.

5. Once you have finished bathing, release the bath plug and allow all of the water to drain out of the bath *before* unlocking and opening the door.

Always ensure that the water has been drained from the bath before opening the door.

Helpful hints to maximise the quality of your bathing experience

- Ensure that the bathroom is warm before using the bath. This will help make your bathing more enjoyable.
- Draping a small towel across your shoulders whilst the bath is filling is a good way to ensure that you are warm and relaxed
- We suggest that, as the bath is filling, you wash yourself working from the toes upwards so that as the water rises you are consequently rinsed off.
- Keep a towel handy so that you can dry yourself as the bath empties.
- The design of your bath allows you to enjoy a greater depth of water than in a conventional bath. If the additional capacity of your bath is fully utilised during bathing, this can place a greater demand on your domestic hot water supply system than a conventional bath. Thus, we recommend that users ensure there is a small

amount of hot water left in reserve when filling the bath so they may top-up their bath with hot water as required during use.

Cleaning and caring for your walk-in bath

Your walk-in bath tub is a high quality, precision-made unit. However, it does require a small amount of regular care and attention from users to ensure that it will continue to offer you many years of quality operation.

Your bath should be cleaned regularly using a soft, damp cloth in conjunction with either a standard cleaning fluid or a *non-abrasive* cream cleaner.

On no account should abrasive, bleach, spirit or ammonia based cleaners be used on the bath.

Chalk mark deposits from the water or mild scratches may be removed from the bath surface with a mild cutting fluid, such as 'T'-Cut™. This should be applied with a soft cloth and rubbed onto the affected area until a satisfactory finish is achieved. The shine in the treated area can be restored using a silicone based polish.

Metal fittings, such as the door hinges and door locking mechanism, may *only* be cleaned with a soft, damp, soapy cloth.

Weekly cleaning of the door seal with a soft, damp, soapy cloth to remove any mould or grime is required to ensure that it remains in good condition. A badly maintained seal can lead to problems in the future.

Looking after your bath door:


- Pressure must never be placed on the door whilst it is not locked shut, as doing so can damage the door hinges.
- When the bath is not in use, the door should be closed but not locked shut. This will prevent the hinges from being strained whilst not allowing the seal to become compressed over time.

Fault finding


In the event that you need to call customer services about your bath, our helpful team will be better placed to deal with your query if you have been able to correctly identify the problem area. To assist with the identification of certain key parts of your bath, we have included a labelled photographic representation of the models covered by this manual on the following pages.

Before calling customer services, please use the appropriate labelled photograph of your bath to identify the problem area.

Tahiti

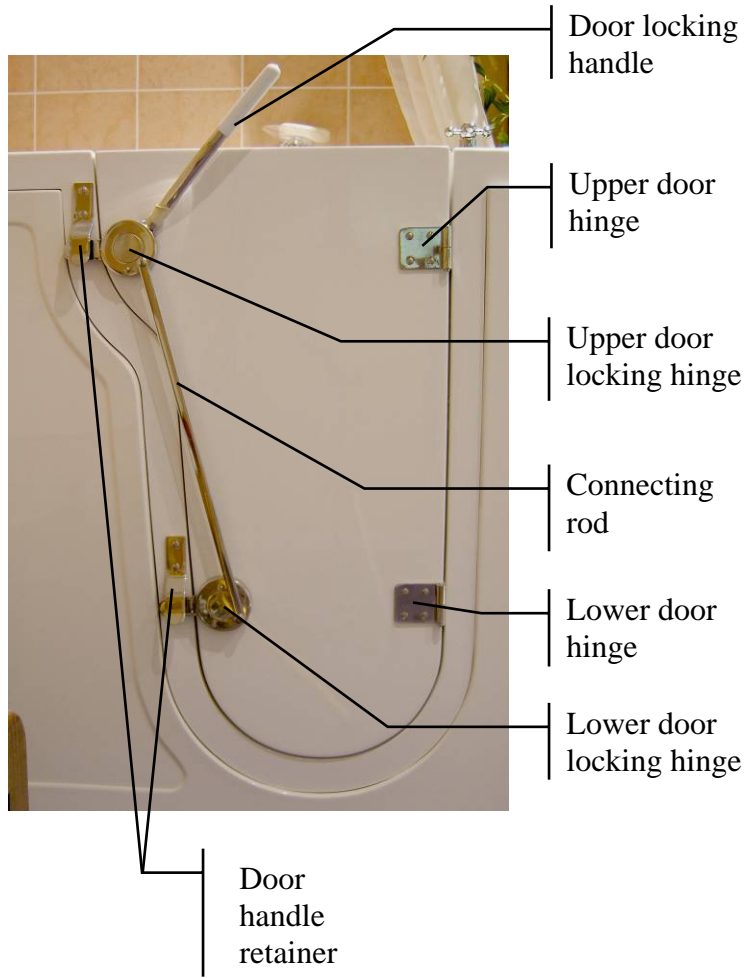

Bermuda/Belgravia

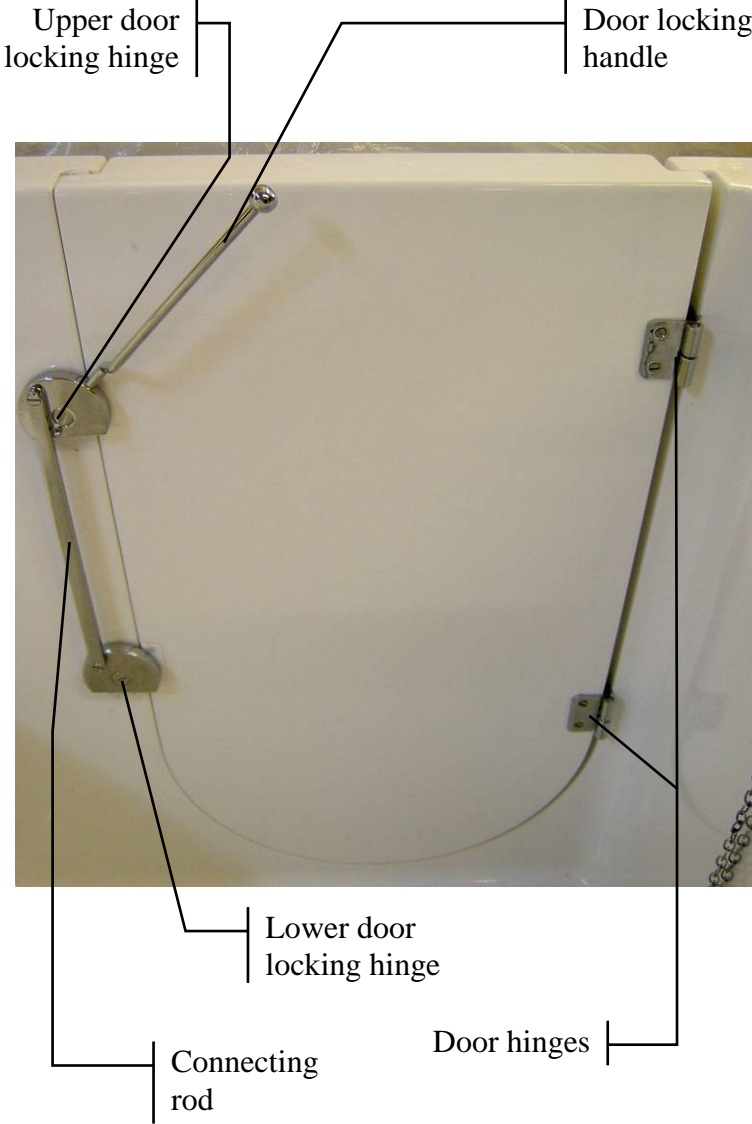

Fiji/Riviera

